


A dark, circular microscopic view of cells, likely from a petri dish, showing various cellular structures and patterns. The background is a deep teal color.

CLEAR X  
BOOSTING GUIDE

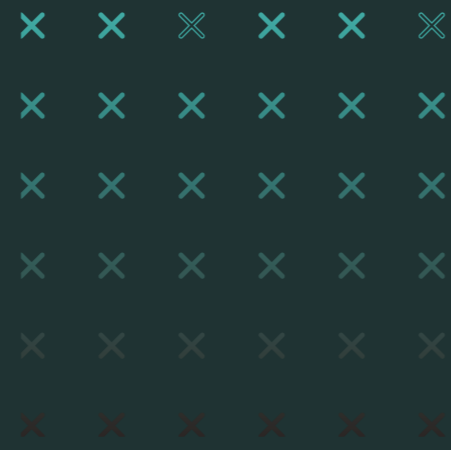

## Your shape-shifting ClearX aligners come to life with a one-of-a-kind boosting process

The revolutionary technology built into each of our ClearX aligners offers two phases of treatment. Your journey to an improved smile begins as soon as your ClearX aligners arrive. Simply wear your ClearX aligners for 7 days and that's shape one complete. Now it's time to add the ClearX touch.

Your shape-shifting ClearX aligners come to life with a one-of-a-kind boosting process, complete with a dedicated smart phone app. The classic booster will engage the ClearX aligner's cutting-edge 4D technology and modify its shape for phase 2 of your treatment.



### 1. Arrival

Receive your tailored ClearX aligners from your doctor.



### 2. Wear

Wear your first set of ClearX aligners for 22 hours a day for 1 week.



### 3. Boost

Activate your shape-shifting aligner with our unique boost process.



### 4. Wear (again)

Wear your ClearX aligners for another 7 days.



### 5. Change

Select your next ClearX aligners and repeat steps 2-4.



### 6. Smile

Once your treatment has finished, show off your new smile to the world.

# CLEAR X time to BOOST!

Here's how to boost your  
**ClearX aligners** in just a few  
simple steps

What you will need...

Kettle | ClearX Aligners | ClearX Classic Booster | ClearX Tweezers | ClearX Aligner Case



## Step 1

### Preparing your aligners

Carefully place your ClearX aligners into the classic booster (open side up) using the tweezers provided.



## Step 2

### Add some heat!

Boil a kettle of water. Once boiled immediately pour hot water into the classic booster up to the max fill line. Ensure that your ClearX aligners are fully submerged.



## Step 3

### Give it some time

Close the lid of your ClearX classic booster to maintain the optimal temperature and wait 10 minutes.



## Step 4

### The big reveal

Open the lid of your classic booster carefully and gently remove your ClearX aligners using the tweezers provided.



## Step 5

### Chill out!

Rest your ClearX aligners in your ClearX aligner case to cool down. Once cooled, your aligners are ready to use.



# HOW DO I KNOW WHETHER THE BOOSTING PROCESS HAS WORKED?

Our unique shape-shifting technology is designed to make your treatment as efficient as possible. After wearing your ClearX aligners for 7 days, you'll perform our simple boosting process to activate the next stage of your treatment.

Here's how to ensure the boost process was a success.

## **Feel the Pressure:**

After the boosting process, you should notice a distinct pressure from the aligner on your teeth. This pressure, and perhaps slight discomfort, indicates that the aligner has successfully shifted to the next stage of your treatment.

## **Check the Fit:**

If the aligner feels the same as it did before the boosting process, it's likely that the process hasn't been successful and needs to be repeated.

## **Troubleshooting**

### **Temperature Matters:**

Ensure you pour the boiled water into the booster immediately after boiling. Waiting too long can reduce the temperature and affect the process.

### **Positioning:**

Make sure your ClearX aligners are placed into your classic booster open side up and are fully submerged in the hot water.

### **Seal the Deal:**

Always close your classic booster lid securely during the boosting process to maintain as much heat as possible.

### **Timing is Key:**

Wait for at least 10 minutes before removing the aligners.

If you've followed all the steps correctly and still don't feel the pressure, it's essential to repeat the boosting process. Remember, the pressure is a sign that your treatment is progressing as planned!





Powered by  
**K line Europe**

SEGRO Park Düsseldorf-Süd  
Section 203 J-K  
Bonner Straße 203  
40589 Düsseldorf-Benrath  
Germany

**Contact us**

[clearx@clearxaligners.com](mailto:clearx@clearxaligners.com)

**Connect with us**

